

•INSTRUCTION / MAINTENANCE MANUAL

STAIR CHAIR

REF	SED001 2 ruote
REF	SED004 4 ruote

1. General warnings

- For correct use of the device, refer to this manual
- The product must only be used by qualified personnel
- Check that the product is delivered to you intact and keep it packed away from heat sources
- The useful life of the device is determined by the wear of non-repairable and / or replaceable parts
- Store in dry places
- Always pay attention to the presence of children

2. Before each use

- Always check the state of wear of the parts in order to guarantee use of the device in total safety for people and things

3. Warnings for use

- Use seat belts when necessary
- Periodically check the stability and sturdiness of the product. Replace it immediately in the presence of breakages.
- Do not use with patients whose weight exceeds that indicated
- If you have any doubts about the use and maintenance of the product, contact the retailer
- To avoid any risks, keep the device out of the reach of children.
- Any accident occurring in relation to the DM in use, supplied by PVS SPA, must be reported to the manufacturer and to the competent authorities.

4. Instructions

- Open the chair (SED001 with 2 wheels - SED004 with 4 wheels)
- Move the patient correctly in the chair (SED001 with 2 wheels - SED004 with 4 wheels)
- Use the appropriate belts to secure the patient
- The presence of handles allows the patient to be lifted by grasping the chair from several sides (SED001 with 2 wheels - SED004 with 4 wheels).
- Maximum capacity of the chair (SED001 with 2 wheels - SED004 with 4 wheels) is 150 Kg.

5. Dimensions (SED001 with 2 wheels - SED004 with 4 wheels)

- Open chair: 910mm (h) x 520mm (w) x 710mm (d)
- Closed chair: 910mm (H) x 520mm (w) x 195mm (d)
- Weight: 8.5 Kg

6. Maintenance

The devices at the time of placing on the market are carefully checked, in case only original spare parts and accessories must be used for repairs.

7. Cleaning and disinfection

Use water and a mild disinfectant; dry away from direct sunlight.

Absolutely avoid using acidic, alkaline or solvent products such as acetone or the thinner.

8. Disposal conditions

In case of disposal of the device, never use the normal waste disposal systems urban solids. Instead, it is recommended to dispose of the device through the common islands municipal ecological systems for the planned recycling operations of the materials used.

9. Warranty

The product has a 24 month warranty against manufacturing defects.

We declare that the device complies with MDR 745/2017 "Medical Devices"



PVS SPA

VIA LEONARDO DA VINCI 18 20051 CASSINA DE PECCHI (MI) ITALY