 <b>SVAS BIOSANA</b>		<b>SCHEDA TECNICA – TECHNICAL DATA SHEET - 033.026</b>	 <b>SISTEMA DI GESTIONE CERTIFICATO ITALCERT UNI EN ISO 9001 UNI EN ISO 13485</b>
Data Emissione	Aggiornamento: 30.10.2019	<b>ASPIDRAIN A</b>	
16/04/2012	Indice: 4		
<b>ADDENSANTE, IN POLVERE, PER LIQUIDI ORGANICI – COAGULATIVE POWDER SUBSTANCE</b>			

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**Description:** *The “Gelid” is a super absorbent polymer powder that turn infected medical liquid waste into a gel-like substance. Mixed with liquid waste, reduces, significantly, the risk of spills from canister, increased safety, convenience and accuracy.*

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**Material:** *Sodium polycrylate cross-linked.*

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*The disposable used to solidify and to reduce the risks associated with manipulation of organic liquids.*

*It can be used to solidify the liquid into the:*

- *surgical and bronchial suction system (canisters or bags),*
- *canister for laboratory liquid,*
- *waste canister,*
- *canister for dialysis liquid,*
- *cases when the liquid is accidentally spilled onto a surface.*

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*Heat sealed pouch. Cardboard box of 25 pouches/box.*

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*ON BOXES ARE REPORTED THE FOLLOWING SPECIFICATIONS:*

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*Store in a cool, dry place, away from heat, light and humidity sources.*

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*The device is manufactured in compliance with UNI EN ISO 9001, UNI EN ISO 13485 and UNI EN ISO 14001, the device meets the essential requirements prescribed by Annex I of Directive 93/42/EEC (subsequent amendments and additions). The entire production cycle follows standard validated methods.*

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*The product must be treated as hospital waste and disposed of according to the regulations in force.*

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*Use is allowed only to qualified personnel.*

*Do not use if the package is not intact.*

*Devices are designed not to be re-used. Their re-use can cause alterations of the performance characteristics of the device compromising the production specifications and consequently the functionality and safety.*

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*Pour the "Gelid" directly into the container (bag or other) containing the liquid and/or blood, seal the container and shake it gently. After the time interval indicated on the package, the product is solidified.*

*In case of spilling on the floor or other surface, pour the "Gelid" directly on the liquid or blood. Remove the solid mass obtained by a spatula and then disinfect the surface.*

Data Emissione      Aggiornamento: 30.10.2019

16/04/2012      Indice: 4

**ASPIDRAIN A****ADDENSANTE, IN POLVERE, PER LIQUIDI ORGANICI – COAGULATIVE POWDER SUBSTANCE**

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