Instructions of Collapsible first aid stretcher DIN13024-2(K)

I. Technical Parameters:

- 1. Product Size (L×W×H): $230 \times 56 \times 14$ cm
- 2. Foldable Size:
- Type K 2: 99x13x17.5cm 3. N.W.:8.5kg
- 4. Loading capacity: less than 150kg



II. How to use:

The folding stretcher is made of high strength aluminum alloy material, with safety belts. This stretcher is intensified stretcher. It can be folded by transverse and lengthwise, with flexible handle and outrigger.

III. Applied for:

Approved by clinic practice, this item is suitable for ambulance, hospital, battlefield, gym, to carry the wounded & patients.

IV. Cautions:

- 1. When carrying patients with this stretcher, be sure to fasten the safety belts.
- 2. In general operation, be careful not to damage the stretcher surface.
- 3. The carrier must inspect the stretcher before any use.

V. Maintenance:

- 1. Keep clean as routine (incl. sterilization).
- 2. Often check if any parts loose or not.

VI. Storage & transportation:

- 1. Store this product in the place of damp proof & non-corrosion.
- 2. Common transporting vehicle can carry this product.