Product name - Plastic Atraumatic Stretcher EN1865

Cod: BAR004

Regulations: The products defined as medical devices are compliant with MDR 745/2017 of 05/04/2017

English

Description	Plastic atraumatic stretcher designed according to rigorous safety and quality standards. The product can be used for magnetic resonance imaging. The structure is conceived for patient immobilization and transfer in emergency and rescue settings.
Materials	Tubular structure: plastic and anodized aluminium Support structure: plastic Hinge joint: aluminium profile
Dimensions	Minimum extension length: 1640 mm Maximum extension length: 2000 mm Folded length: 1180 mm Width: 445 mm Folded stretcher thickness: 85 mm Weight: 7.5 kg Loading capacity: 159 kg
Instructions for use	Open the stretcher completely and ensure the male/female joint locks correctly before use. Position the patient following emergency

	<p>immobilization procedures and fasten the three safety straps securely. Use the length adjustment group when needed to adapt the device to the patient. After use, clean and disinfect according to internal protocols and store in a dry place.</p>

Product name - Plastic Atraumatic Stretcher EN1865

Cod: BAR004

Regulations: The products defined as medical devices are compliant with MDR 745/2017 of 05/04/2017

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