## Product name - BASKET STRETCHER

Cod: BAR018

**Regulations: The products defined as medical devices are compliant with MDR 745/2017 of 05/04/2017**

### English

#### Description

This basket stretcher is intended for the recovery of patients in environments that are difficult to reach, where other devices cannot operate. Typical use includes water rescue, recovery on rough terrain, rapid recoveries, and transfers where protection against lateral impacts is required. The device is made in a single piece and is not divisible. The perimeter frame provides excellent rigidity and a secure handhold for handling. It is equipped with rings for securing hooks, a perimeter rope for attaching accessories, rounded lower corners to reduce friction during dragging on uneven ground, and a mattress to improve patient comfort. Compatible with all Spencer spine boards.

#### Dimensions

Length: 2170 ± 10 mm  
 Width: 620 ± 10 mm  
 Height: 185 ± 10 mm  
 Number of handles: 12  
 Materials: PE, PP, aluminium, steel  
 Weight (without standard accessories): 13.5 ± 0.5 kg  
 Loading capacity: 278 kg

#### Instructions

Place the patient in the stretcher following approved rescue procedures and secure them using the supplied safety straps. Use the footrest when

## **Kit contents**

needed to improve positioning stability. Handle and lift the stretcher only by trained personnel, using the designated handholds and attachment points. Before and after each use, check the integrity of the frame, straps, rings and accessories. Clean and store the device in a dry, protected place.

Standard supplied contents:

- Footrest
- Two-piece safety straps (3 straps supplied)

Compatible / optional accessories:

- Floating structure
- Adjustable harnesses
- Fixed harnesses
- Towing straps
- Carrying bag
- IV pole for basket stretcher

**Product name - CIVIÈRE PANIER**

**Cod: BAR018**

**Regulations: The products defined as medical devices are compliant with MDR 745/2017 of 05/04/2017**

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