Product name - mobileRAPID WRIST – Automatic wrist blood pressure monitor with Bluetooth®

Cod: SFI000

Regulations: The products defined as medical devices are compliant with MDR 745/2017 of 05/04/2017

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English

Description

Automatic wrist digital blood pressure monitor with Bluetooth® connectivity.

Technical specifications

Method: oscillometric (wrist). Range: pressure 40–255 mmHg; pulse 40–199 bpm. Pressure sensor: piezoresistive. Accuracy: pressure ± 3 mmHg; pulse $\pm 4\%$ of reading. Inflation: automatic pump. Deflation: automatic valve. Auto power-off: 60 s. Power: 2×AAA 1.5 V (LR03). Operating environment: 10–40 °C; 15–85% RH. Storage: –20–50 °C; 15–85% RH; Atmospheric pressure: 70–106 kPa. Cuff: wrist circumference 12.5–21 cm, BF type. Digital display 46×52 mm; resolution 1 mmHg; memory 2×90 with date/time; unit 91×74.5×15 mm; weight 90 g (without batteries); pouch. Bluetooth®: module SH878801; 2.402–2.435 GHz; 3.3 V supply.

Instructions

Wear the cuff on the bare wrist (display facing up) 2–3 cm above the joint. Keep the wrist at heart level, press START/STOP and remain still and silent until completion. Readings (SYS/DIA/PULSE) can be stored; for connectivity open the companion app and follow Bluetooth® pairing steps. Clean with a soft cloth; avoid immersion, shocks and extreme temperatures.

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