Product name - Sanaphon NEW – Palm-style aneroid sphygmomanometer with stethoscope (Riester)

Cod: SFI237

Regulations: The products defined as medical devices are compliant with MDR 745/2017 of 05/04/2017

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English

Description

Palm-style aneroid sphygmomanometer with stethoscope – Sanaphon NEW (Riester).

Technical specifications

Construction: sturdy, lightweight aluminium casing with corrosion protection; stainless-steel spoon-shaped handle. Connection: push-in tube connection on the right side of the gauge. System: aneroid manometer with specially hardened copper-beryllium membrane, virtually non-wearing. Scale range: 0–300 mmHg, 64 mm linear scale, easy to read; maximum error tolerance ± 3 mmHg.

Overpressure: membrane withstands up to 600 mmHg. Bulb: latex inflation bulb; precision air-release valve, finely adjustable and wear-free; micro-filter protects the release valve and measuring system. Cuff: pull-through strap Velcro cuff with integrated chest piece; wide range of sizes. Accessories: stethoscope, padded nylon zipper bag, personal blood pressure recording booklet. Also available as latex-free (LF).

Instructions

Place the cuff on the bare upper arm 2–3 cm above the elbow crease. Position the stethoscope over the brachial artery. Inflate with the bulb to ~30 mmHg above estimated systolic; release slowly with the valve and read values using the auscultatory (Korotkoff) method. Fully deflate, clean and store in the padded pouch.

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