**Product name - NEOSPLINT NEOPRENE SPLINTS**

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## English

### Description

Rigid moldable neoprene splints with internal aluminium core. They provide effective immobilisation thanks to their ability to be shaped and are fitted with Velcro closures for quick fastening. SplintArt splints are simple and practical devices designed to immobilise limbs. The set includes a complete range of splints for the treatment of sprained or fractured limbs; they can be shaped as needed and, once applied according to the clinical requirement, provide excellent rigidity. Thanks to their light weight and adaptability, the complete set or individual items can be carried anywhere while taking up very little space. As they are made of neoprene, they also help maintain body temperature.

### Dimensions / Available models

STE122: forearm

STE123: arm

STE124: ankle / elbow

STE125: leg

STE126: wrist

STE127: complete 5-piece set with bag

### Materials

Neoprene structure coupled with MTP D90 and black cotton jersey.

Rigid cold-mouldable aluminium core.

Blue and orange Velcro fastening and adjustment system.

### Intended use

Sprained limbs.

Fractured limbs.

### Application method

1. Place the injured limb on the SplintArt.
2. Shape the metal core as required.
3. Adjust and close using the Velcro straps; the orange Velcro indicates the upper side.  
To return the SplintArt to a straight position, straighten the metal core by hand.

### Indications

Post-traumatic or post-operative conditions of the limbs in which effective support and/or protection of the skeletal structure and muscle groups is required. If the device is used in the presence of skin lesions, protect the lesion with appropriate bandaging; in such cases, do not use the device in direct contact with the lesion. Medical device to be applied under medical supervision. Radiolucent. Latex free. Non-sterile. Single patient use.

### Contraindications / Warnings

Disorders of the venous, arterial or lymphatic system. In case of suspected pathology, consult a physician before use. Use for prolonged periods only under medical supervision.

### **Storage**

Store in a dry place away from sunlight, heat sources and dust. The product has no expiry date, but the expected life of materials and components is 10 years.

### **Washing**

Hand wash in water at 30°C with soap or neutral detergent.

Rinse thoroughly.

Do not wring.

Leave to dry flat.

Do not iron.

### **Packaging**

Primary: 1 piece in PVC bag.

Secondary: box of 5 pieces.

### **Product compliance / Reference standards**

Directive 93/42/EEC.

Class I medical device.

ISO 9001:2008.

Italian Ministry of Health Medical Device Register no. 2370831.

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