

Extractor Instruction



MD

Ingredients (see figure 1)

- Plastic box
- Extractor
- 4 cups, the diameter is 2.5 cm, 2 cm, 1.8 cm, 1.5 cm respectively.

Functions

- Provides more suction (750 millibars) and remove more venom.
- No cutting, no scalpel necessary
- Safe for children
- Safer and more convenient than taking a scalpel or other knives.
- Also helpful in treating the bites and stings, homets, mosquitoes
- Recommended medically as the only acceptable first aid device for snakebites

How to use the extractor?

1. **install the cup(see figure 2):** insert a cup into the mouth of extractor and rotate the it until fixed.

2. **Absorbing venom:** In order to achieve maximum suction (750 millibars), the suction device should be as vertical as possible (see figure 3). The suction is created by simply thrusting the plunger downward using the thumb and two fingers.(see figure 4).

Make sure that you always use the extractor with one hand and press hard on the plunger.

This is the only way to achieve the best performance.

Be careful to choose the right cup and put it vertically. Do not use excessive pressure to avoid damage to extractor and cups.



FOR YOUR ATTENTION! Be sure always to keep the cup clean and hygienic. Pay attention to the quality of the extractor.

FIGURE 1



FIGURE 2

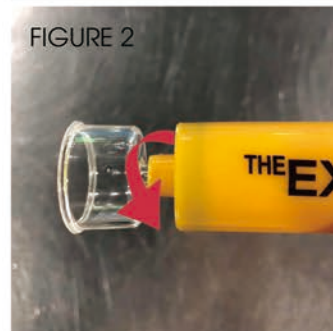


FIGURE 3

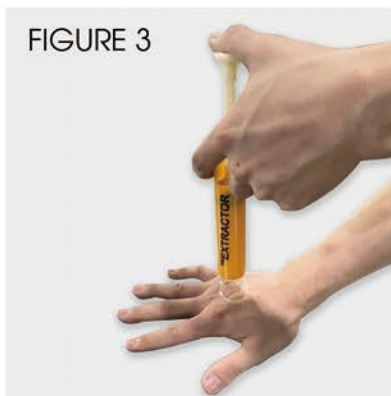


FIGURE 4



CARE & CLEANING OF THE EXTRACTOR

- Do not wash the extractor or get it wet! This may destroy the vacuum-creating mechanism in the extractor.
- You should wash and sterilize the cups after each use but do not wash the extractor.
- Thoroughly cleaning the cups with soap and water is usually adequate. Treating with alcohol is best. Do not boil the cups in water.

In case of improper use of the product, we have no liability.

We declare that the device complies with MDR 745/2017 "Medical Devices.

It is necessary to report any serious accident occurring in relation to the DM provided by PVS SPA to the Manufacturer and the competent authorities.

