Product name - Mercury-free sphygmomanometer with LED display

Cod: SFI247

Regulations: The products defined as medical devices are compliant with MDR 745/2017 of 05/04/201

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English

Description

Mercury- free aneroid sphygmomanometer with column and LED display. Manual inflation via bulb and manual deflation with air- release valve. The column flashes when pressure exceeds 315 mmHg. Suitable for adults and pediatric use. Supplied with PVC bulb and bladder, adult nylon cuff (25–38 cm) with D- ring.

Technical features

Pressure range: 0–300 mmHg (± 3 mmHg). Pulse range: 30–200 bpm ($\pm 5\%$). Power: 3 $\times$ AA batteries, DC 4.5 V (not supplied). Weight: 584 g (without batteries and cuff). Protection: Type BF applied part; IPx21 rating.

Instructions

Insert batteries observing polarity. Connect the cuff to the device. Place the cuff on the bare upper arm, 2–3 cm above the elbow crease. Inflate with the bulb to 20–30 mmHg above the expected value, then slowly release pressure using the valve and read the values on the column/display. Wait 2–3 minutes before repeat measurements. Clean with a soft slightly damp cloth; do not immerse. Store in a dry place, away from sunlight and heat. Intended use: non- invasive blood pressure measurement. Diagnostic use only.

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